



HEALIX MEDICO

110 Manish Ind Est. No. 4, Navghar Road, Vasai (E), Dist Thane, Pin 401210
Tel: +91-250-2390707

Healix Whitelite is the perfect combination of simplicity, durability and effectiveness. It is designed specially to suit the Indian conditions. The light produces blue light which makes the bleaching material function quickly and effectively in a short period of time. Its body is ergonomically designed to suit the hand grip and it is also longer than conventional lights providing a good reach while usage.

Components and Principles:

The Healix Whitelite is based on ray radiation principles and is designed to cause faster dissociation of Hydrogen Peroxide molecules with the radiation of energy.

Basic Parameters:

Power Adaptor:
Input Voltage: 220 V AC
Operating Voltage: 5.7V DC
Operating current: 800mA
Light Source Wavelength: 420 – 480nm

Operating Method:

Programming Method:

1. Switch on the power switch.
2. Press the "SET" button
3. The display will start blinking
4. Using the "UP" and "DOWN" keys select the cycle time (eg. 20 mins for Contrastam)
5. Press "SET" again.
6. The cycle time is now stored in memory till you change it again.

Operating Method:

1. Plug in the power cord.
2. Aim the light source at the fixed position
3. The working distance must be within 3 inches from the fixed position
4. Press the button to start the cycle of the light.
5. The light would automatically switch off at the end of the cycle.

Safety Precautions:

1. The light source has to shoot directly in the material when it is in clinical use. Improper radiation may affect the curing result
2. The machine should be kept away from water
3. The users must be qualified for the profession concerned.
4. Do not use on patients who have biological reaction symptoms.

Maintenance:

Store the curing light in its original packing when not in use.
Keep in a clean & dry place.